




Sectional Matrix System

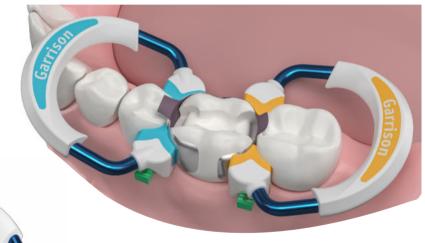







Why should you use a Sectional Matrix System?



Strata-G[™] Sectional Matrix System


- + Rings produce optimal tooth separation for excellent, tight contacts
- + Bands with pronounced curvature for anatomically correct contacts
- + Contacts at the height of contour
- + Operator-friendly retaining system
- + No post-processing necessary
- + 26 years of experience in sectional matrix systems combined in the Strata-G sectional matrix system

Valid until 31.12.2023 or while stocks last. Errors and modifications reserved. The colours in the images may vary from the original product. All prices are non-binding recommended prices. Our Terms and Conditions apply. All prices subject to VAT.





Now faster and more predictable!

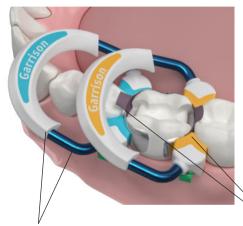

Faster, less frustrating Class IIs

The Strata-G[™] rings, bands and wedges combine for a 270° Seal[™] for ultimate flash control. The rings won't spring off the tooth thanks to their retentive tips, and the amazing green Wide Preparation ring makes tough wide preps no problem.

Reliable, tight contacts

Rings made of drawn-wire nickel titanium plus PEEK produces consistent separating pressure longer!

Ultra-durable PEEK–increases

toothseparating pressure and longevity.

Easier, more predictable contours

Marginal ridge enhancements built into the matrix bands AND rings help provide that finishing flair!

Strata-G[™] forms a 270° Seal[™] for ultimate flash control.



Perfect for dentists who want to get a complete set of equipment.

SG-KSH-11 Price: £719

		V J A	J
All Strata-G [™] sets h	nave a 60-day money-back guarantee.**	\searrow	\sim
Part No.	Description		Price
SG-KSH-11 Professional Set	 1x Strata-G[™] Ring short blue, 1x Strata-G[™] Ring long orange, 1x Strata-G[™] Ring wide green, 50x Strata-G[™] matrix bands with non-stick coating, 5 sizes (10x each SG050, SG100, SG175, SG200, S 50x Strata-G[™] Firm matrix bands, 5 sizes (10x each FXH100, FXH150, FXH175, FXH 80x Strata-G[™] wedges, 4 sizes (20x each SGYL, SGBL, 1x Ring placement forceps FXP01, 1x matrix band ho 1x 5-in-1 multi composite instrument TN009, 10x VariStrip[™], 5x Margin Elevation matrices 	, SGOR, SGGR),	£719

NEW!

SG-KSH-11

TRATA-G







SG-KS-00

Enough bands/wedges included in the kit to make the most of the 60 day money back guarantee.

SG-KS-00 Price: £495

All Strata-G[™] sets have a 60-day money-back guarantee.**

Part No.	Description	Price
SG-KS-00 Starter Set	 1x Strata-G™ Ring short blue, 1x Strata-G™ Ring long orange, 1x Strata-G™ Ring wide green, 100x Strata-G™ matrix bands with non-stick coating, 5 sizes (20x each SG050, SG100, SG175, SG200, SG300), 80x Strata-G™ wedges, 4 sizes (20x each SGYL, SGBL, SGOR, SGGR), 1x Ring placement forceps FXP01 	£495



Perfect for avid users of Garrison Dental Solutions' previous products.



SG-KS-40

STRATA

SG-KS-40 Price: £430

All Strata-G[™] sets have a 60-day money-back guarantee.**

Part No.	Description	Price
SG-KS-40 Standard Set	 1x Strata-G[™] Ring short blue, 1x Strata-G[™] Ring long orange, 1x Strata-G[™] Ring wide green, 50x Strata-G[™] matrix bands with non-stick coating, 5 sizes (10x each SG050, SG100, SG175, SG200, SG300), 40x Strata-G[™] wedges, 4 sizes (10x each SGYL, SGBL, SGOR, SGGR), 1x Ring placement forceps FXP01 	£430

5 **If you don't like the set, simply send it back to us via your depot and you will receive a full credit.




Choose a Strata-G[™] sectional matrix system set and ...

... you will receive your Strata-G[™] set including the 60-day money-back guarantee. During this time you can test all components of the set. If you are not satisfied, simply return the set to us via your depot and you will receive a full credit.

... Benefit from more than 26 years of world-leading innovation in sectional matrix systems

Rings can now be placed in reverse or butterfly placement

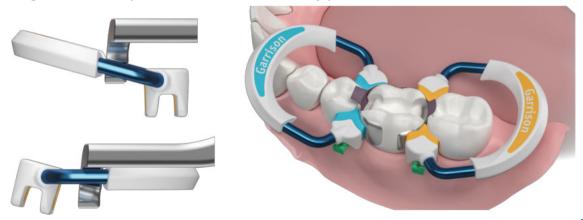

Sectional Matrix Rings

Carrison

The green Strata-G™ ring for wide cavities

- Holds even when cusps are missing.
- Especially good for large cavities.
- The wide silicone tips prevent the matrix from being pressed into the cavity.

Part No. SG600 (2 pieces / pack)

The orange Strata-G™ ring - ring metal now made of Nitinol

- Drawn nickel-titanium wire strong and elastic at the same time.
- Ultra-Grip™ retention tips prevent all Strata-G™ rings from slipping off.
- Holds even on large molars.

Part No. SG500 (2 pieces / pack)

The blue Strata-G[™] ring is now even easier to open

- Optimal adaptation to the marginal ridge due to improved StrataBond™ silicone.
- Especially suitable for short/ malpositioned teeth and in pediatric dentistry.
- Also holds on a ruberdam clamp or distally on the canine. Part No. SG400 (2 pieces / pack)

Part No.	Description	Price
SG600	Strata-G™ Ring Wide Prep green, 2 rings/pack	£213
SG500	Strata-G [™] Ring Tall orange 2 rings/pack	£213
SG400	Strata-G [™] Ring Short blue 2 rings/pack	£213
SGR01	Strata-G™ Ring Pack (1of each ring)	£312
FXP01	Ring Placement Forceps	£87











Longer lasting by design

Advanced materials and design combine to produce our longest lasting, most predictable separator ring ever!



Strong tooth separation yet easier to open

Nitinol drawn-wire nickel titanium is aligned at the molecular level for superior performance yet requires less effort to open. Garrison's proprietary StrataSet[™] treatment of the Nickel-Titanium wire maximizes the longevity of this advanced material. The rings are now easier to open and hold their tension longer.

More uses

The PEEK ultra-durable reinforcement on the back of the ring allows Strata-G™ to retain a higher percentage of initial strength for much longer than non-reinforced rings. Proprietary StrataBond™ silicone/PEEK tips withstand hundreds of applications.

Improved anatomy - Unbeatable grip

Enhanced marginal ridge guides and supports matrix bands.

trataBond™ silicone for unsurpassed flash control.

Ultra-Grip[™] retention extensions provide tenacious grip and are gentler to the papilla.







Easy removal of the matrix bands thanks to the nonstick coating, which reduces adhesion to the composite by an impressive 92%.



Full band kit including all band sizes, 400 bands (80x each SG050, SG100, SG175, SG200, SG300)

SGB01 £340

Full band kit including all band sizes, 200 bands (40x each SG050, SG100, SG175, SG200, SG300)

SGB02 £180

Improved marginal ridge design due to stronger precontouring at the occlusal band part.







Anatomy top to bottom

Five sizes of super-thin stainless steel in both dead-soft and firm ensure you have the best matrix band to restore every case.



Optimized marginal ridge

trata-G[™] Bands provide shape and guidance for developing the marginal ridge of Class II restorations without blocking your access.

ProfileBooster™ subgingival apron

Achieving a natural emergence profile in challenging cervical areas is easy with Strata-G[™] Bands - it's built right in!



- Non-stick coating and color coded
- Easy placement tabs

- Dead-soft and only 0.038mm thin
- NEW pedodontic size!









Full wedge kit including all wedge sizes, 400 wedges (100x each SGYL, SGBL, SGOR, SGGR)


£189 SGK4

Full wedge kit including all wedge sizes, 200 wedges (50x each SGYL, SGBL, SGOR, SGGR) £101

SGK4-M

design and rounded edges glide over the papilla with less trauma.

Smooth tunnel

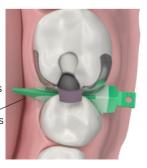





Flexibility for a better cervical seal

Strata-G™ Wedges utilize Soft-Face™ materials and advanced design to promote a better fit.

No back-out

Soft retentive fins smoothly fold down during wedge insertion and spring back when clear of the interproximal space, locking Strata-G[™] Wedges in place.

A better seal by design

The inverted V shape sits lower and seals deeper interproximally while flexible side walls and Soft-Face™ TPE over-mold allow Strata-G™ Wedges to easily adapt to concavities and irregularities.



Stack them up

The smooth tunnel design easily accommodates stacking of wedges. This feature is extremely helpful in cases of gingival recession or when simultaneous wedging from the buccal and lingual is desired.



Sectional Matrix System

Technique for Class II restorations utilizing Strata-G[™] Soft Face[™] Rings

- 1. Place rubber dam. Isolation of the area helps to prevent contamination from blood and saliva. As with many dental procedures, aspiration of small parts is a valid concern.
- 2. Prepare tooth with conservative Class II preparation.
- 3. Place Strata-G[™] sectional matrix band. Select a matrix band that most closely approximates the occlusogingival height of the tooth. The Grab-Tab should be oriented toward the occlusal margin and may be bent over the adjacent tooth after placement. (figure 1)
- 4. Insert Strata-G[™] wedge. Keep a finger on the matrix band to prevent the wedge from dislodging the band during insertion. Wedge firmly, a high viscosity resin packed into the preparation may displace the band away from the tooth. (figure 2)
- 5. Apply the Strata-G[™] Soft Face[™] ring retainer. Select a retainer ring that is appropriate for the embrasure and tooth being restored. Spread it with the Garrison Premier Ring Placement forceps and place it over the wedge. (figure 3)
- 6. Very Important: Push the matrix band firmly into contact with the adjacent tooth in the desired contact area.
- 7. Use your preferred etching, disinfecting and bonding technique following manufacturer's instructions. Apply your preferred composite resin following manufacturer's instructions.
- 8. Remove the ring, wedge and band.
- 9. Contour and polish the restoration.

Stacking of Rings & MOD Restorations

The orange tipped and blue tipped versions of the Strata-G™ Soft-Face Rings are angled to allow stacking of rings for performing M.O.D.s and multiple-tooth restorations. (figure 4)

Alternatively, the Garrison Premier Ring Forceps (FXP01) allow for reverse, or 'butterfly', placement of the distal ring





Figure 2



Figure 3



Figure 4

Contact form By fax: +49 (0) 2451 971-410

NEW! Order easily and conveniently, without registration via our online catalogue:

https://catalogue.garrisondental.net/strataguk/

Simply scan the QR code!

By phone: +49 (0) 2451 971-409 By UK phone: 0800 00 64 90

E-Mail: info@garrisondental.net

Online catalogue: https://catalogue.garrisondental.net/strataguk/

 \Box **Order:** (will be supplied by your dental depot)

Quantity	Part No.	Unit price in £	Total price in £
	SG-KSH-11 - Strata-G [™] Professional Set	£719	
	SG-KS-00 - Strata-G [™] Starter Set	£495	
	SG-KS-40 - Strata-G™ Standard Set	£430	
Practice _	D	ental Depot	-
No. Street	in	I	
Town/City D		ate/Signature	
Postcode			
Phone			
	al Solutions • Carlstrasse 50 • D-52531 Uebach-Palenberg • (51 971-409 • Fax +49 (0)2451 971-410 • E-Mail: info@qarriso		risondental.com





Garrison Dental Solutions Carlstrasse 50 52531 Uebach-Palenberg Germany UK Phone 0800 011 2738 Phone +49 (0)2451 971-409 Fax +49 (0)2451 971-410 E-Mail: info@garrisondental.net Web: www.garrisondental.com





https://catalogue.garrisondental.net/strataguk/

