

## Impress-E Wash Instructions for Use

### IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.  
**SPECIALLY FORMULATED FOR USE IN DENTISTRY**

### IDENTIFICATION

Product Name	Impress-E Wash 2 x 50ml
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

### PRODUCT CODE - REF

PP010/1282 Impress-E Wash Light Body 2 x 50ml

PP010/1283 Impress-E Wash Medium Body 2 x 50ml

### INDICATIONS FOR USE

Silicone indicated for the two-step putty/wash, one-step putty/wash and sandwich techniques. The impression can be electroplated.

### INSTRUCTIONS FOR USE

Working and Set times for Product:

Work: 2 min. Minimum Time in the mouth: 1min 30s. Total Set/Work time: 3min 30s.

1. Dry the areas to be reproduced and prevent further inflow of liquid into them. Load the cartridge into the dispenser.
2. After removing the cap, expel a small amount of silicone from the two nozzles, moving the lever several times.
3. Insert the mixing cap into the cartridge and if necessary, place the small oral tip on this for perfect and precise gingival extrusion. Actuate the lever forcefully so as to express the Impress-E Wash from the cap, loading the mixed material into the impression tray.
4. Insert it into the mouth.

It is important to check that the extruded silicone is evenly coloured. To preserve any product remaining in the cartridge correctly, leave the mixing cap mounted until the next use. Before applying a new mixing tip, ensure that the silicone comes out of the nozzles regularly. If the cartridge remains unused for a long period, it may become blocked by hardened silicone, which must be removed with a fine instrument before a new mixing tip is mounted.

### WARNINGS

The working and set time can be influenced by:

- Variations in the temperature of the material, the environment and/or the operator's hands during mixing (higher temperatures reduce them while lower temperatures prolongs them);
- Changes in the speed and/or force employed during mixing;
- Humidity;
- Contact of the silicone with latex rubber dam or gloves or with components of such material possibly present on the teeth during the impression procedure (vinyl gloves are recommended).



Do not invert lids of the containers and use the corresponding coloured scoop for each paste. Close the containers well after use. To eliminate traces of saliva or blood and guarantee the optimal adhesion of the wash impression to the preliminary impression, rinse under running water and dry well. The impression can be kept without any special devices. Cast the model in plaster at least 30 minutes after withdrawal from the mouth.

For disinfection, immerse the impression for 5 minutes in a solution of glutaraldehyde (concentration 0.5%-2%) or sodium hypochlorite (concentration 0.1%-1%). Rinse the impression under running water after disinfection.

### **Caution**



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

**STORAGE**

Store in a cool, dry place.

**BATCH CODE/EXPIRY DATE**

See the print on the bag. Do not use the product after this date.



Perfection Plus Ltd  
6 Westwood Court  
Brunel Road  
Hampshire  
SO40 3WX, UK  
[www.perfectionplus.com](http://www.perfectionplus.com)



Perfection Plus EU Ltd  
The Black Church  
St Mary's Place.  
Dublin, Ireland,  
D07 P4AX  
[Regulatory@perfectionplus.com](mailto:Regulatory@perfectionplus.com)



Advena Ltd  
Tower Business  
Centre  
2nd Floor Tower  
Street  
Swatar, Malta

