SeptoPlus Mandrel / SeptoDiscs Mandrel

Accessory device for fixing different finishing and/or polishing systems in contra-angle. Available in two systems: mandrel by pressure (SeptoPlus Mandrel) and pop-on mandrel (SeptoDiscs Mandrel).

Use: SeptoPlus Mandrel: Insert the chuck of the contra-angle handpiece. Install the finishing/polishing device on the mandrel. SeptoPlus Mandrel: Compatible with SeptoPolisher and SeptoWheels PX, SeptoDiscs Mandrel: Compatible with SeptoDiscs. **Warnings, precautions and restrictions:** Professional use only. Use of the mandrel or discs with similar products is of the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operating field. If there is possibility to any of its components, the product must not be used. Wear protective goggles to shield against particles (dentists and patients). Use of a rubber dam is highly recommended to avoid the potential ingestion of dental materials or other debris. Discs must be handled with care and control to avoid injury to the patient. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Cleaning: Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amilase, Protease, Lipase and Carboxidase) and with pure pH range between 6.5 - 7.5. Prepare the dilution solution according to the manufacturer's instructions. The water should be potable. For best effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the product and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the piece with absorbent paper or disposable wipe. **Sterilization:** The mandrel should be autoclave (126°C, 16min) after each patient. **Reprocessing Limits:** Repeated processing has minimal effect on the mandrel due to its composition, this is a reusable device. SeptoPlus Mandrel: The manufacturer guarantees up to 50 reuses. SeptoDiscs Mandrel: The manufacturer guarantees up to 240 reuses. **Inspection:** Carefully inspect the mandrel to ensure that all debris has been removed. Observe damage / breakage of the product that would prevent proper operation and do not use in this case.

Composition: SeptoPlus Mandrel: Stainless steel. SeptoPlus Mandrel: Nickel plated metal.

Storage: Store below 30°C (86°F), in a dry place, protected from light. **Connection with contra-angle:** The SeptoPlus Mandrel / SeptoDiscs Mandrel is manufactured in compliance with specifications ISO 1797:2017.

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Warnung, Vorsichtsmaßnahmen und Einschränkungen: Nur für den professionellen Gebrauch. Nur für den professionellen Gebrauch. Professional use only. Use of the mandrel or products is the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operating field. If there is possibility to any of its components, the product must not be used. Use of a rubber dam is highly recommended to avoid the potential ingestion of dental materials or other debris. Discs must be handled with care and control to avoid injury to the patient. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Ostrzeżenia, środki ostrożności i ograniczenia: Wyłącznie do użytku profesjonalnego. Wyłącznie do użytku profesjonalnego. Professional use only. Use of the mandrel or products is the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operating field. If there is possibility to any of its components, the product must not be used. Use of a rubber dam is highly recommended to avoid the potential ingestion of dental materials or other debris. Discs must be handled with care and control to avoid injury to the patient. Contact with soft tissues (skin, gingiva, mucosa) may result in injury to the tissue. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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